





# Vacuform® Patient Positioning System

The optimised, advanced surgical bedding concept.


The secret of perfected adaption to the body by Vacuform® surgical mattresses is microballs of Styrofoam, which are stabilised by vacuum to give a lasting, completely supportive form. Reducing the diameter of the filter balls by a fifth enables more precise adaption of the surgical mattress to the body. The MicroSpace system significantly increases the feeling of physical comfort for the patient compared to conventional fillings.

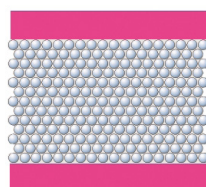

Arrangement of the Vacuform® MicroSpace balls without vacuum.

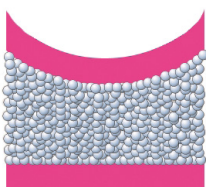


Arrangement of the Vacuform® MicroSpace balls under vacuum.

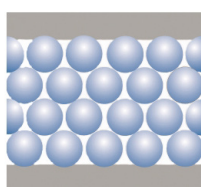

Arrangement of conventional Styrofoam balls without vacuum.

The individual, ergonomic body adaptation of Vacuform® surgical mattresses helps prevent pressure points leading to acute decubital pressure injuries and paresis. The support pressure of the patient is evenly distributed, and thus minimised. The microcirculation needed by the skin is unaffected and the patient has a noticeable feeling of lightness and warmth.

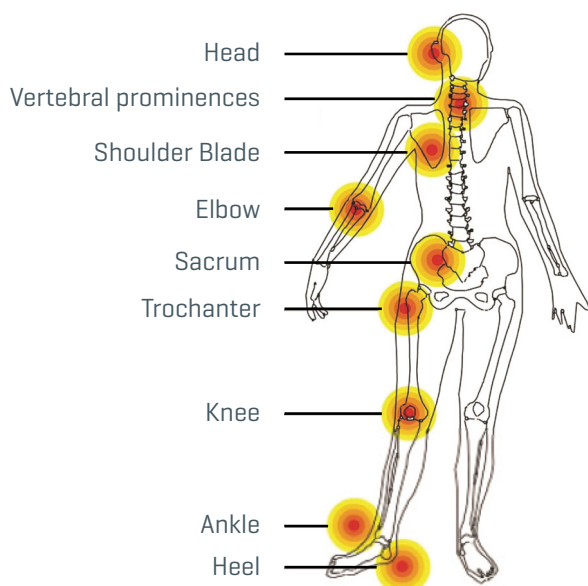

Parts of the body at risk from decubitus

- Further optimised pressure relief.
- “Weightless” feeling (like a water bed) despite stable immobilisation.
- Surface almost untextured.
- Latex-free, radiolucent and suitable for MRI and CT.



MicroSpace filling in the evacuated state.



Conventional filling in the evacuated state.

## HOW DOES VACUFORM® WORK?

Visit [www.hipac.com.au](http://www.hipac.com.au) or scan the QR code to view our tutorial video.




## NEW VACUFORM® VACUUM PUMP

In case there is no vacuum system in the operating room, we now offer a metal vacuum pump with foot stand.

## VACUFORM® SURGICAL MATTRESSES



Vacuform® Surgical Cushion with Cutout  
400[L] x 350mm[W]  
**CODE:** 114988

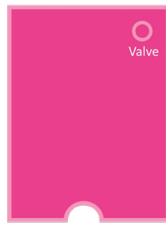

Vacuform® Surgical Mattress with Cutout  
1000[L] x 750mm[W]  
**CODE:** 114994

**MOST  
POPULAR  
SIZE**



Vacuform® Surgical Mattress with Cutout  
1000[L] x 1000mm[W]  
**CODE:** 114990



Vacuform® Surgical Mattress with Cutout  
1250[L] x 750mm[W]  
**CODE:** 114993

## VACUFORM® VACUUM PUMP

In case there is no vacuum system in the operating room, we now offer a metal vacuum pump with foot stand.

**CODE:** 143618



## VACUFORM® NON-SLIP FOIL

Prevents slippage of the surgical mattress in extreme positions. The sheeting is easy to clean and reusable.

Piece - 1m[L] x 400mm[W]

**CODE:** 114975

Roll - 9m[L] x 400mm[W]

**CODE:** 114974



## VACUFORM® PARTS



Vacuform® Repair Kit

**CODE:** 139067



Vacuform® Replacement Valve

**CODE:** 117309

For more information, please scan the QR code or visit [www.hipac.com.au](http://www.hipac.com.au). To request a quote or place an order, contact our Customer Service Team on 1800 75 93 93 or [info@hipac.com.au](mailto:info@hipac.com.au).



Details shown in the flyer are correct at time of print and are subject to change without notice.

## Hipac also offers:

### Stainless Steel Trolleys




### UVC Disinfection Robots



### Clinical Furniture



### At Risk Patient Protection

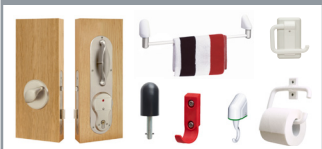

Check out our full range of products by visiting [www.hipac.com.au](http://www.hipac.com.au).



We're here for life

PO Box 797, 36 Long St. Goulburn NSW 2580, Australia

P. +61 2 4823 0000 E. [info@hipac.com.au](mailto:info@hipac.com.au)

SYDNEY | MELBOURNE | BRISBANE | ADELAIDE | PERTH

## ABOUT HIPAC

We are an Australian healthcare company that guarantees to stand behind all our medical products, both manufactured and imported, because we're here for the life of our products and clients.

**hipac.com.au**

ABN 27 600 353 688